



The Bionix SecureFit Arm/Wrist Support is intended to be used for the positioning and re-positioning of patients undergoing or receiving a course of external beam radiation therapy for the treatment of cancer and other diseases.


Caution: Federal law restricts this device to sale by or on the order of a licensed physician.

## COMPONENTS

- |                                       |   |
|---------------------------------------|---|
| A:: Arm Cup Assembly                  | F:: Expansion Handle Assembly           |
| B:: Secure Fit knob Assembly          | G:: Horizontal Jam Nut & Spring Plunger |
| C:: Vertical Jam Nut & Spring Plunger | H:: Horizontal Post                     |
| D:: Stem Assembly                     | I:: Horizontal Adjustment Collar        |
| E:: Vertical Adjustment Collar        |   |

## ASSEMBLY DIAGRAM

Diagram coloring used to demonstrate component differentiation. Not actual product color.

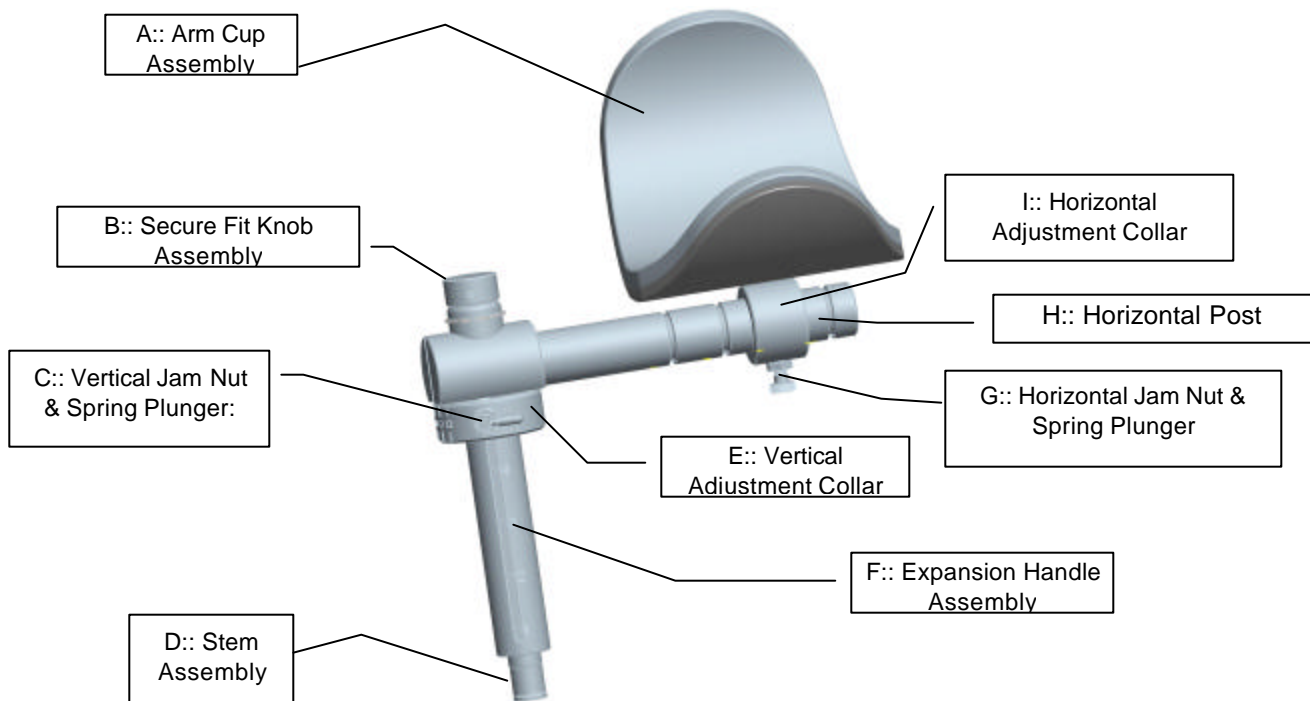

## OPERATING INSTRUCTIONS

1. Insert Stem Assembly (D) into desired perimeter holes labeled A through H on the TorsoBoard.
2. Turn the Secure Fit Knob (B) clockwise to secure the device in place.
3. Pull the Vertical Jam Nut & Spring Plunger (C) to desired height indexing point.
4. Slide the Horizontal Post (H) down to insert the two placement pins to desired position (A, B, C, or D)



Operating Instructions for the  
BIONIX SecureFit Arm/Wrist Support  
Product #: RT-5043

5. Pull down on the Horizontal Jam Nut & Plunger (G) to desired position (7, 8, 9, 10) and the desired tilt (X, Y, Z).
  6. Position patient arms into Arm Cup Assembly (A).
  7. NOTE: Be sure to record the settings used for each patient.
- To Remove:
8. Turn counter-clockwise on the Secure Fit Knob Assembly (B) to loosen and remove the Arm/Wrist Support.

### CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

Notice: Do not use alcohol based cleaners on this product.

### WARRANTY

Your Bionix product is warranted to be free from defects in workmanship and materials, under normal use, for a period of 3 years from the date of original purchase from Bionix. Bionix' sole obligation under this warranty shall be, at Bionix' option, to replace the product or part with a comparable product or part, repair the product or part, or if neither repair nor replacement is reasonably available, Bionix may, in its sole discretion, refund to the customer the purchase price paid for the product or part.

Bionix Radiation Therapy  
5154 Enterprise Blvd.  
Toledo, OH 43612  
Toll Free: 800-624-6649  
Fax: 800-455-5678  
[www.BionixRT.com](http://www.BionixRT.com)