

The Bionix BackStop Immobilizer is intended to be used for the positioning and re-positioning of patients undergoing or receiving a course of external beam radiation therapy for the treatment of cancer and other diseases.

Caution: Federal law restricts this device to sale by or on the order of a licensed physician.


COMPONENTS

A:: Base Assembly
 B:: Indexing Stud

C:: Pad Assembly
 D:: TorsoBoard (Figure 1)

ASSEMBLY DIAGRAM

Diagram coloring used to demonstrate component differentiation. Not actual product color.

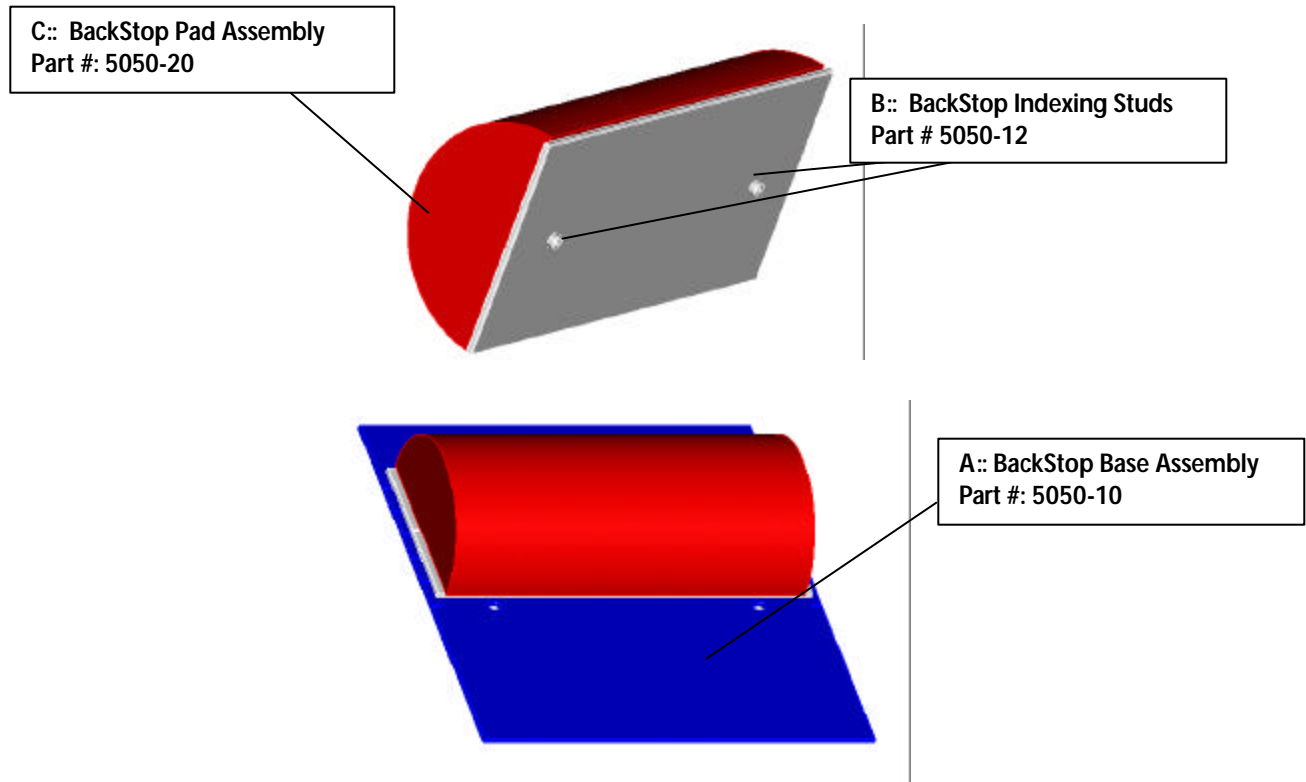

OPERATING INSTRUCTIONS

Backstop Indexing Base Attachment Instructions

1. Prepare TorsoBoard (D) without transition pad on the treatment table (see Figure 1).
2. Align the Velcro strips on the bottom of the Base Assembly (A) with the Velcro strips on the TorsoBoard (D), and attach.

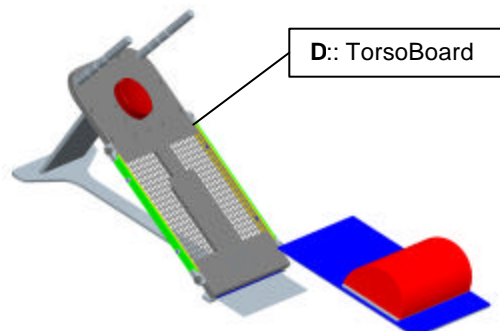


Figure 1



Backstop Pad Attachment Instructions

1. Using the hash marks on the edges of the Base Assembly (A) and the Pad Assembly (C) align the Indexing Studs (B) with the desired indexing holes.
2. Insert the Indexing Studs (B) into the holes on the Base Assembly (A).

CLEANING INSTRUCTIONS

To clean, wipe thoroughly with water based antiseptic cleaner or foam. Allow to dry before next patient use.

Notice: Do not use alcohol based cleaners on this product.

WARRANTY

Your Bionix product is warranted to be free from defects in workmanship and materials, under normal use, for a period of 3 years from the date of original purchase from Bionix. Bionix' sole obligation under this warranty shall be, at Bionix' option, to replace the product or part with a comparable product or part, repair the product or part, or if neither repair nor replacement is reasonably available, Bionix may, in its sole discretion, refund to the customer the purchase price paid for the product or part.

Bionix Radiation Therapy
5154 Enterprise Blvd.
Toledo, OH 43612
Toll Free: 800-624-6649
Fax: 800-455-5678
www.BionixRT.com