



CLEANING INSTRUCTIONS FOR BIONIX OTOCLEAR IRRIGATION PRODUCTS

General Care – All OtoClear Irrigation Products:

The best way to maintain any of the OtoClear Irrigation products is to rinse the unit with clean water after each use and then let it air dry completely. Most problems occur when the OtoClear Irrigation products are stored in wet areas or kept sealed for long periods of time with liquid inside.

OtoClear Gentle Touch Irrigation Tip:

- The OtoClear Soft Tip is a single use medical device and is NOT to be reused.

OtoClear Spray Wash System (including OtoClear Ear Basin):

- General cleaning of the OtoClear Spray Wash System can be accomplished by washing the components in a solution of mild detergent and hot water. Then, completely rinse the components with clean water, drain and allow them to completely dry.
- Disinfection of the OtoClear Spray Wash System can be accomplished by using CidexPlus® 28 Day Solution, Cidex® OPA Solution or Revital-Ox™ Resert® High Level Disinfectant, and following the manufacturer's directions for use.

OtoClear Adapter Wands:

- General cleaning of the OtoClear Adapter Wands can be accomplished by washing them in a solution of mild detergent and hot water. Then, completely rinse the OtoClear Adapter Wands with clean water, drain and allow them to completely dry.
- Disinfection of the OtoClear Adapter Wands can be accomplished by using CidexPlus® 28 Day Solution, Cidex® OPA Solution or Revital-Ox™ Resert® High Level Disinfectant, and following the manufacturer's directions for use.

Waterpik® Water Flosser WP-440

(Note: The bullet points are excerpts from the WP-450 Instruction Manual)

- Never immerse the unit in water. Wipe the exterior when necessary with a soft cloth and mild non-abrasive cleanser.
- Reservoir may be washed with soapy water or in the top rack of dishwasher.
- Before cleaning unplug from the electrical outlet.