


Enteral Feeding Tube DeCloggers®

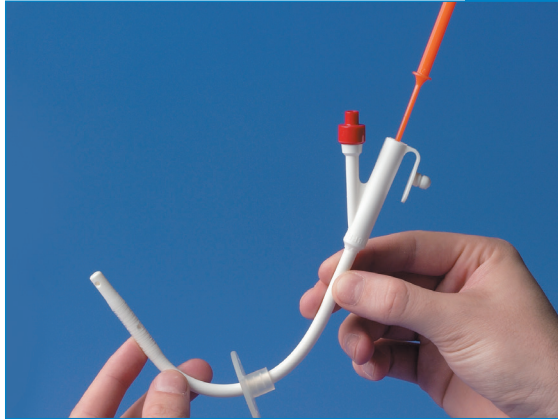

Bionix DeClogger helps healthcare provider's clear obstructed G, J, and PEG tubes in less than two minutes to restore medication delivery and reduce untimely tube replacement.

► PROBLEM:

Inevitably, over time, G, J, and PEG tubes clog preventing patients from receiving nutrients and medication.

► SOLUTION:

DeClogger, a safe, flexible threaded device that bores through occlusions to quickly restore nutrition and medication to patients with obstructed enteral feeding tubes.

► RESULTS:

Enteral feeding tubes that are free of obstruction in less than two minutes and capable of providing patients with the nutrients and medications that they need.



©2016 Bionix Development Corp.

“
The DeClogger has greatly reduced my need for tube replacement.
”
- Scott Polsky, MD



Scan in the QR code with your web enabled camera phone to view our DeClogger Video.



Obelis s.a • Boulevard Général Wahis 53 • 1030 Brussels, BELGIUM
Tel: +(32) 2.732.59.54 • Fax: +(32) 2.732.60.03 • E-Mail: mail@obelis.net




Bionix Development Corp. • 5154 Enterprise Blvd. Toledo, OH, 43612, U.S.A.
Phone: 800-551-7096 • Fax: 800-455-5678 • www.Bionix.com

Phone: 800.551.6810
www.Bionix.com

BIONIX® GOING-BEYOND ►

Enteral Feeding Tube DeCloggers®

PRODUCT OFFERING

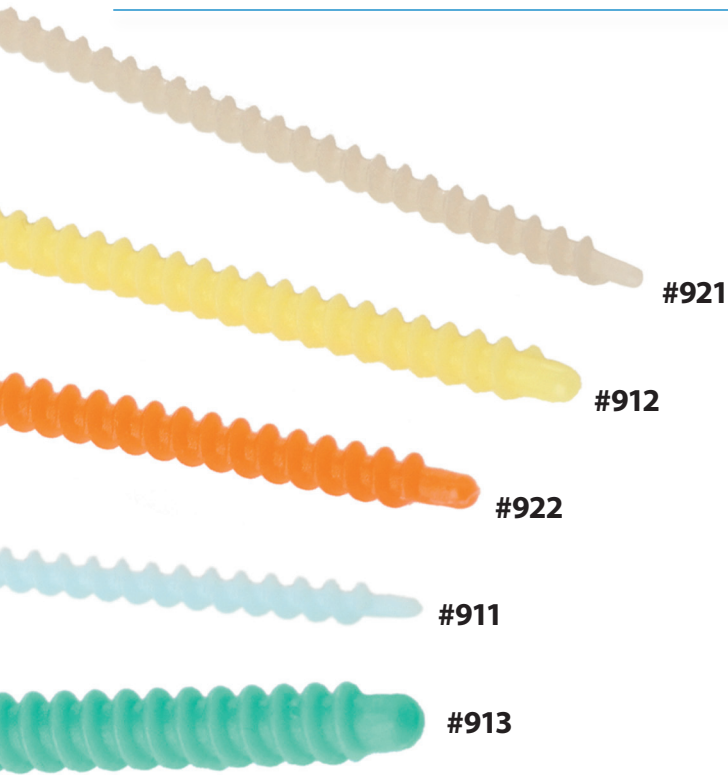

Gastrostomy or Jejunostomy

Product Number	Product Size	Length
911 – Blue	14 – 16 French	39.5 cm
912 – Yellow	16 – 18 French	39.5 cm
913 – Green	20 – 22 French	39.5 cm

Peg Type or Replacement Type

Product Number	Product Size	Length
921 – Brown	14 – 16 French	21.5 cm
922 – Orange	18 – 24 French	21.5 cm

DeCloggers are packaged 10 per box.

USAGE INSTRUCTIONS

	<p>Over time, residue from enteral solutions and medications clog the tube, putting the patient at risk.</p>		<p>The DeClogger is inserted down the tube where slight twisting action is applied and the build-up or clog is dislodged and cleared.</p>		<p>Within two minutes, the build-up or clog can be cleared and the tube restored to its normal free-flowing condition.</p>
--	--	--	---	--	--



GOING BEYOND >

Phone: 800.551.6810
www.Bionix.com

